



I.V. HOUSE®
Protection Over and Above




I.V. House UltraDressing®
Intravenous Site
Protectors



Item 730M

Available in Small, Medium and Large Sizes
Designed To Fit Either Hand

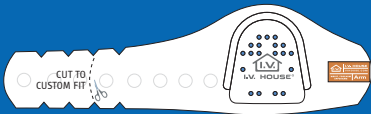


Item 730Arm

Adult and Pediatric Patients:

- Wrist, Forearm, Antecubital Space or Upper Arm

Neonates and Infants:

- Hand, Wrist, Forearm, Antecubital Space or Upper Arm
- Foot and Saphenous Placement

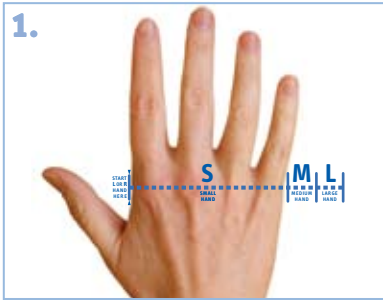

Adult Hand Directions

Each dispenser box includes twenty-four hand measurement charts. Simply place the patient's hand on the left margin of the chart to determine if they need a small, medium, or large size I.V. House UltraDressing.

The measurement is taken from the proximal interphalangeal joint of the 1st and 4th finger (knuckle to knuckle) and will assign the correct size needed.

Insert the IV catheter and securely attach an IV extension set or IV tubing. Secure IV site with transparent dressing, tape, or adhesive anchor. Loop the IV tubing away from the catheter hub and toward the IV fluid source.

Make sure the loop of tubing or extension set is small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDressing.

(Step-by-step directions for IV site taping are also available at www.ivhouse.com)

Position the I.V. House UltraDressing over the looped tubing and catheter hub.

Wrap the I.V. House UltraDressing around hand.

Close with Velcro®.

Inspect the IV site according to nursing practice guidelines: release the closure tab under the label and lift the dome to visualize the entire IV site. After viewing the entire IV site, re-close the device.

www.ivhouse.com
800.530.0400






I.V. HOUSE
Protection Over and Above



I.V. House UltraDressing®
Intravenous Site
Protectors



LATEX FREE

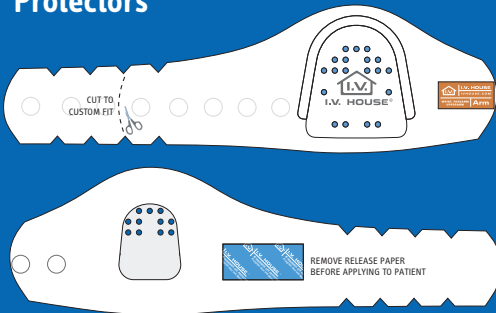


Item 730Arm

Adult and Pediatric Patients:

- Wrist, Forearm, Antecubital Space or Upper Arm




Neonates and Infants:

- Hand, Wrist, Forearm, Antecubital Space or Upper Arm
- Foot and Saphenous Placement

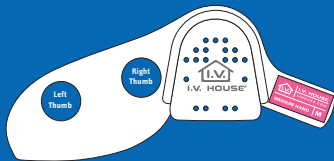

Items 730S, 730M and 730L

- Designed To Fit Either Hand

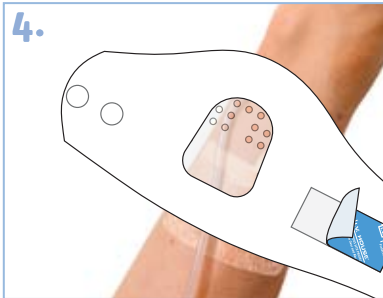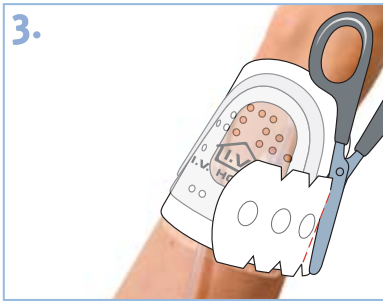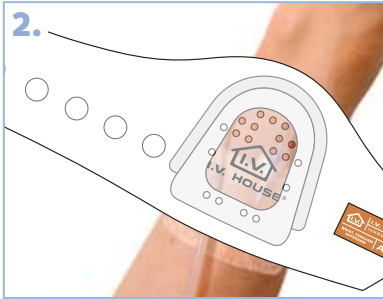

Adult Arm Directions

Insert the IV catheter and securely attach an IV extension set or IV tubing. Secure IV site with transparent dressing, tape, or adhesive anchor. Loop the IV tubing away from the catheter hub and toward the IV fluid source. Make sure the loop of tubing or extension set is small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDressing.

(Step-by-step directions for IV site taping are also available at www.ivhouse.com)

Position the I.V. House UltraDressing over the looped tubing and catheter hub. To determine appropriate fit, wrap the long end of the fabric comfortably around the arm overlapping the orange label to determine how much fabric to cut off.

Cut off excess fabric at the appropriate notch. Be sure to leave 2 Velcro® loop coins for the closure on the long end of the fabric. Always cut off extra fabric.

Any extra fabric under the dome would decrease the visibility of the site and is not recommended.

Turn the device over and remove the blue release paper from the strip of soft cloth surgical tape. The purpose of this gentle tape is to keep the device from sliding down the patient's arm.

Place the I.V. House UltraDressing over the IV site. Apply the fabric around the arm, gently pressing the tape into place. Close the device with the Velcro® hooks and loops. Always allow enough stretch for two fingers to fit comfortably under the fabric.

Inspect the IV site according to nursing practice guidelines: release the closure tab under the label and lift the dome to visualize the entire IV site. After viewing the entire IV site, re-close the device.

www.ivhouse.com
800.530.0400



Adult IVHUD Directions

800-530-0400

Learn More: visit our website: www.ivhouse.com

Instructive Videos / Usages / Directions / Product Details and Ordering Information